

Instructions For Use

1. Place teeth into a cup of hot (almost boiling) water for 1 minute. This will make them flexible and ready to custom fit.
2. Using a mirror, firmly position the teeth over your existing teeth. Press them firmly against your teeth and gently pat the impression material with your thumbs behind your real teeth.
3. Gently ease the teeth out of your mouth after 30 seconds and set them in a glass of cold water to cool. The impression material turns white when it is hardened.
4. The impression is now moulded to the shape of your teeth and can be used as and when desired.



Teeth
Veneer

Adhesive

Please Read All Instructions Thoroughly Before Use

Tips: For Adult Use Only.

For Cosmetic Purposes Only. "This is a Temporary Solution" Hard but pliable plastic.

Warning: Please do not use it to eat or sleeping.